DA TRACOE care Trakeakompressor

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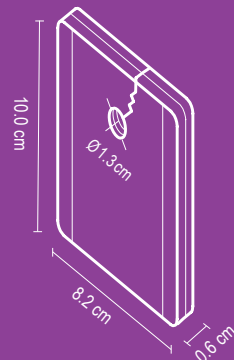
REF 957

EN TRACOE care Tracheal Compresses

coated, two-ply, with zigzag slit, for neonates, children, adolescents and adults

TRACOE *care* Tracheal Compresses are used to provide padding between the tracheostomy tube and the tracheostoma. The compresses help to absorb secretions and protect the skin beneath the neck flange.

The coating reduces adhesion to the skin. Apply the compress, so that the shiny coated side is in contact with the skin.

The slit in the compress is oriented towards the chin.

Change the compress at least once a day or several times a day, if secretion is heavy.

Warning: Using the compress for a long period of time or re-using it may give rise to a rash. In the event of an allergic reaction such as itching, reddening or other signs of hypersensitivity, remove the compress and consult a doctor. Avoid wrinkling the compress when applying it. When applying or removing the compress, make sure that the tracheostomy tube does not slip out of place. Do not use for sterile wound treatment.

When the compresses are used together with other medical devices, follow the respective instructions for use.

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